 **Sogeva**®

STAY QUALITY, STAY SOGEVA

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SOGEVA, SAFETY AND QUALITY IN THE INDUSTRY SINCE 1996.

For almost 30 years we have continued to establish ourselves as a company manufacturing and trading in medical and non-medical devices, especially in the field of professional protection and sterilization, in dental and medical world.

Our commitment is to renew ourselves year after year by focusing on improving quality, always with an eye on environmental sustainability, thus consolidating our 5 product lines:

- **STERILIZATION PACKAGING LINE**
- **STERILIZATION CONTROL LINE**
- **STERILE DRAPES LINE**
- **ICE PACK LINE**
- **DENTAL DISPOSABLES LINE**

Thanks to direct control over in-house production, we offer the possibility to produce customisable products in terms of format, type, size and packaging according to customer requirements.

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● Disponibile in Tyvek

SOGEVA sterilization reels are produced with the best certified materials: high grammage medical paper (60 g/m²) and a double layer of polyethylene/polypropylene film (55g/m²), in compliance with MDR (EU) 2017/745 specially designed to ensure high antibacterial protection for medical sterilisation. State of art processes: a constant seal during sterilization; extreme ease of opening without fibre release or tearing; proper maintenance of sterility.

STERILIZATION COLOR CHANGE:

Steam: light blue to green

Ethylene Oxide: pink to orange

Formaldehyde: green to yellow.

● **6 months sterility** from the date of production under suitable storage conditions and upon validation of the sterilization and sealing process

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Gusseted reel is especially studied for the sterilization of items that need a bigger capacity in order to satisfy the different needs of packaging.

● **6 months sterility** from the date of production under suitable storage conditions and upon validation of the sterilization and sealing process

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mm 75xh25mmx100 mt | 08 Pcs/Carton
mm 100xh50mmx100 mt | 06 Pcs/Carton
mm 150xh50mmx100 mt | 04 Pcs/Carton
mm 200xh55mmx100 mt | 04 Pcs/Carton
mm 250xh65mmx100 mt | 02 Pcs/Carton
mm 300xh80mmx100 mt | 02 Pcs/Carton
mm 400xh60mmx100 mt | 02 Pcs/Carton

PERSONALIZZABILE CUSTOMIZABLE

In base alla richiesta del cliente
According to the customer's request

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Heat-sealable pouches made of white medical paper with high grammage (60g/m²) and double layer of blue polyethylene/polypropylene film (55g/m²), in compliance with MDR (EU) 2017/745 and EN 868-5. With three uniform seals we guarantee; extreme ease of opening without fibre release or tearing; proper maintenance of sterility. Equipped with chemical indicators for steam or EO gas (ethylene oxide) sterilization. The ISO 11140-certified inks are water-based and non-toxic, and provide excellent color change of toning.

● **6 months sterility** from the date of production under suitable storage conditions and upon validation of the sterilization and sealing process

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mm 50x250 | 1.000 Pcs/Carton
 mm 75x200 | 1.000 Pcs/Carton
 mm 75x250 | 1.000 Pcs/Carton
 mm 75x300 | 1.000 Pcs/Carton
 mm 100x250 | 1.000 Pcs/Carton
 mm 100x300 | 1.000 Pcs/Carton
 mm 150x250 | 1.000 Pcs/Carton
 mm 150x300 | 1.000 Pcs/Carton
 mm 150x400 | 1.000 Pcs/Carton
 mm 200x300 | 1.000 Pcs/Carton
 mm 200x400 | 1.000 Pcs/Carton
 mm 250x400 | 1.000 Pcs/Carton
 mm 250x450 | 1.000 Pcs/Carton
 mm 250x500 | 1.000 Pcs/Carton
 mm 300x500 | 1.000 Pcs/Carton
 mm 300x600 | 1.000 Pcs/Carton
 mm 400x500 | 1.000 Pcs/Carton
 mm 500x600 | 1.000 Pcs/Carton

PERSONALIZZABILE CUSTOMIZABLE

In base alla richiesta del cliente
 According to the customer's request

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Gusseted pouches are especially studied for the sterilization of items that need a bigger capacity in order to satisfy the different needs of packaging.

● **6 months sterility** from the date of production under suitable storage conditions and upon validation of the sterilization and sealing process

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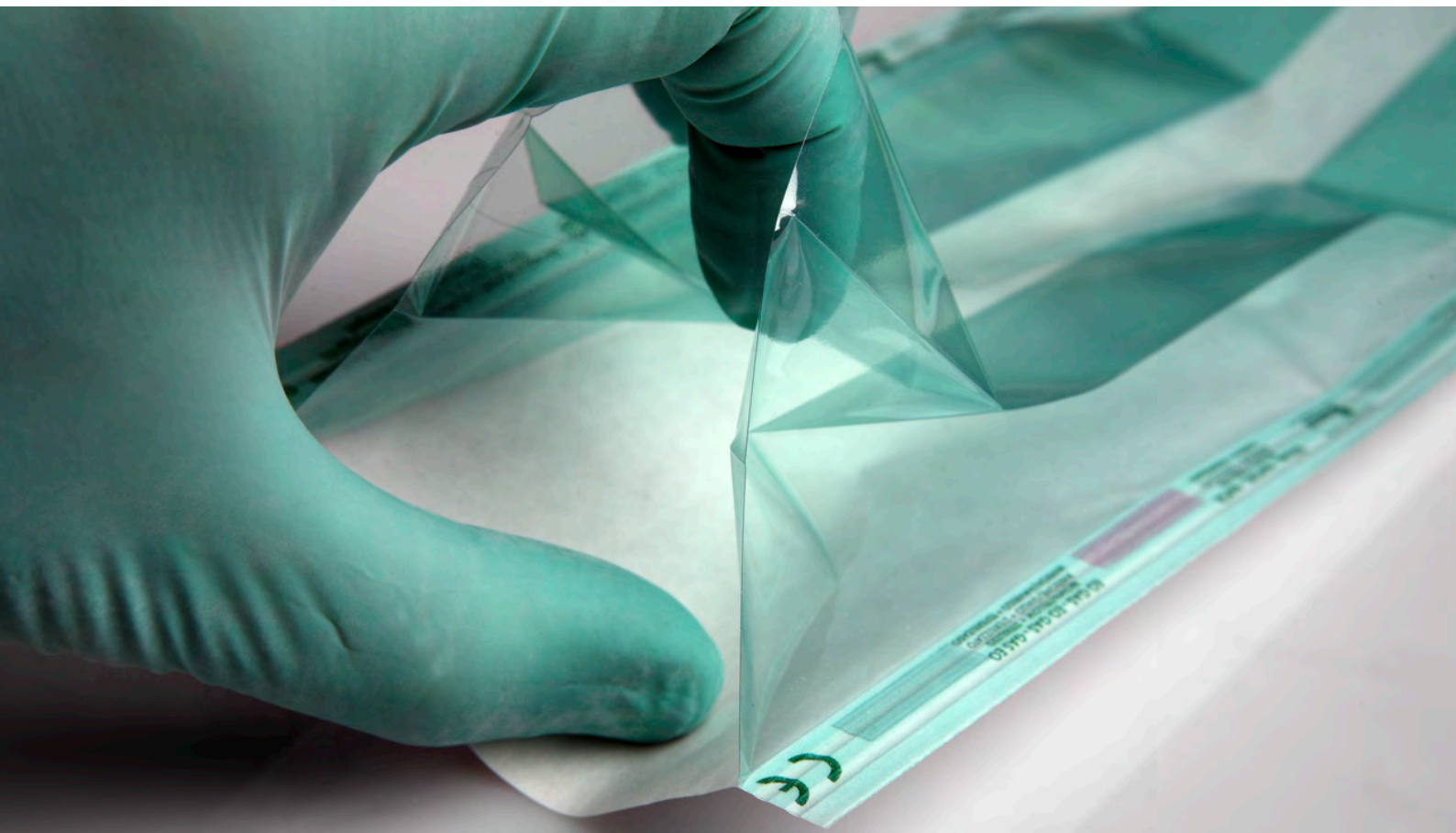
Autosealing pouches provide very strong sealings (multiline) and equally divided. Autosealing pouches have the adhesive band made by the innovative system "finger-lift" which allows an easier use. **Duo-check system:** Double sterilization indicator, internal and external. It ensures high control in the sterilization process.

◆ **6 months sterility** from the date of production under suitable storage conditions and upon validation of the sterilization and sealing process

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mm330x440 - 200 Pcs/Carton | 04 Boxes/Carton

PERSONALIZZABILE CUSTOMIZABLE

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According to the customer's request

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Sogeva SMS wrapping paper consists of four nonwoven polypropylene fabric layers and can be used either as inner or outer wrapping for medical devices and trays. The spunbond layers (S) provide good filtration properties, tensile strength and are fluid repellent while the meltbown layers (M) ensure strong bacterial barrier properties with very small porous size. SMS wraps as single or double wrapped are suitable for steam, ethylene oxide, hydrogen peroxide plasma and irradiation sterilization methods. The memory-free feature before and after sterilization, ensures good aseptic presentation.

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Sogeva Crepe paper is produced from pure white kraft pulp and consists entirely of primary virgin fibre. It is used for wrapping medical material during steam and EO sterilisation cycles and offers an excellent barrier service, is moisture-resistant and stable during the sterilisation cycle by minimising water condensation.

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STERILIZATION CONTROL PRODUCTS LINE

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Disposable pack BOWIE&DICK is designed to carry out the Bowie & Dick test, check the correct removal of air from the inside of the porous loads in defined sterilizers "to high vacuum for porous loads" operating with sub-atmospheric cycles or trans-atmospheric. It is used for the daily monitoring of steam autoclave with pre vacuum operating at 134° C - 137° C for 3-3,5 min. and 121°C for 15 min.

CERTIFICAZIONI CERTIFICATIONS

Conforme alle norme ISO 11140-3 ; 11140-1 ; 11140-4 Classe 2 Conforming to norms 11140-3 ; 11140-1 ; 11140-4 Class 2

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96 pz per scatola 96 pcs | carton

PERSONALIZZABILE CUSTOMIZABLE

In base alla richiesta del cliente
According to the customer's request

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Helix PCD Test)is used as a daily test for pre-vacuum hospital sterilizers and for bench top steam sterilizers Class B for steam penetration and inert gases. The Helix PCD test is an excellent test to detect even the smallest amounts of inert gases and steam penetration capabilities. Supplied in a KIT with 1 tube 1.5 m long and 250 sterilization indicators.

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Class 4 Steam Indicator Strips Class 4 Steam indicator strips are designed to be used in steam sterilizers operating at 134°C -3.5 minutes. The indicator strips can be used in every pack and will provide assurance of steam penetration into the packs. The Sogeva indicator strips can be divided into two separate pieces for economical use.

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Sogeva Class 5 Chemical Integrators can be used inside each load to monitor time, temperature and steam exposure conditions and can provide the necessary sterilization assurance. Class 5 integrator mimics the ability of a biological indicator at three separate points during the sterilization process because it will monitor both time and temperature without requiring incubation as does a biological indicator.

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The indicator tapes allow the correct sterilization cycle to be assessed. Resistant to high temperatures and humidity, they have high adhesion to various types of cloth, metals, plastics, non-wovens and papers; the high conformability allows easy manual handling.

- **STEAM STERILIZATION** - indicator changes from light green to medium/dark green
- **DRY STERILIZATION** - indicator changes from green to brown
- **EO STERILIZATION** - indicator changes from light yellow to dark green/blue

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Sterile Surgical Drapes are used during surgery to prevent contact with unprepared surfaces and to maintain the sterility of environmental surfaces, equipment and the patient's surroundings, and the hygiene of the personnel and ensure the safety against contamination risk. They are made of layers of laminate and absorbent material to provide 100% impermeability on the bottom layer and high level absorption of fluid from the top layer.

STERILE DRAPES LINE

- Surgical drapes
- Surgical drapes with adhesive
- Fenestrated drapes

CERTIFICAZIONI CERTIFICATIONS

Dispositivo medico - classe IS
Medical device - Class IS

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Teli chirurgici con lato adesivo | Surgical drapes with adhesive side

75x90mm
100x150mm
75x115mm

Teli defenestrati | Fenestrated drapes

100x150mm
75x115mm

COLORI COLORS

azzurro light blue

PERSONALIZZABILE CUSTOMIZABLE

In base alla richiesta del cliente
According to the customer's request

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- TNT
- PE

Disposable instant ice for instant cold indicated in cases of trauma, to soothe pain and prevent bruising or swelling. Sogeva's Instant Ice is distinguished by its unique formula that reaches a cold temperature of up to -4 degrees in seconds and maintains it.

TWO TYPE AVAILABLE:

- Non woven
- Plastic

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24 sacchetti per scatola
24 bags | carton

DIMENSIONI SIZE

14x18 cm
14x24 cm

PERSONALIZZABILE CUSTOMIZABLE

In base alla richiesta del cliente
According to the customer's request

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Sterile, disposable surgical aspiration cannula made by plastic material and sterilized by EO. Blutip suction cannulas are proven accessories for drying the surgical field, particularly in minimally invasive oral surgery and particle aspiration.
- Can be sterilized by steam in autoclave.

CERTIFICAZIONI CERTIFICATIONS

Dispositivo medico - classe IIA
Medical device - Class IIA

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20pz imbustati singolarmente e un adattatore n°2
20pcs single packed and 1pcs adapter n°2

DIMENSIONI SIZE

Puntale tip:

- Ø 1,2 mm micro
- Ø 2,5 mm standard

Cannula cannula:

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PERSONALIZZABILE CUSTOMIZABLE

In base alla richiesta del cliente
According to the customer's request

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The saliva ejectors is manufactured from pvc with an insert of galvanized or copper-plated iron. The filter part is produced from polyethylene in the removable cap type. The device is disposable and it has to be thrown away after use.

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Universal shaped tips for surgical aspirator, made of plastic material that can be sterilized in autoclave up to 134°C.

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Universal shaped tips for surgical aspirator, made of plastic material that can be sterilised in autoclave up to 134°C.

CERTIFICAZIONI CERTIFICATIONS

Dispositivo medico classe IIA
Medical device class IIA

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DIMENSIONI SIZE

- ADAT.1** Adattatore 11,0 mm | terminali 16,0mm
Adapter 11.0 mm | terminals with 16.0mm
- ADAT.2** Adattatore/riduttore 11,0mm e 4,8mm
| terminali 16,0mm Adapter/reducer 11,0mm and 4,8mm | terminals 16.0 mm
- ADAT.3** Adattatore | 4,80 mm terminali | 11,0 mm
Adapter 4.80 mm | terminals 1.0 mm

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Tidespon is a sterile biocompatible, implantable, absorbable gelatin sponge with hemostatic effect, produced from pharmaceutical purified porcine gelatin used to take control of bleeding during the operation. When it is applied to the hemorrhage site, it adheres to the bleeding surface and absorbs blood or fluids approximately 35 times of its own weight. Tidespon is absorbed completely within 3-4 weeks when implanted into tissue, however; the amount of usage, depending on the applied field to absorb may change. Sterility: Gamma Rays (R).

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The protective disposable aprons in roll in paper/plastic are made by a sheet of paper with high absorbent power coupled with a sheet of waterproof polyethylene. **Light blue pocket version available upon request .**

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Color availability can change by aprons size.

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Professional towels are composed of 1 or 2 plies of pure cellulose laminated and embossed with a layer of high-density polythene film. The perfect absorbency and impermeability of the product ensures high patient comfort.

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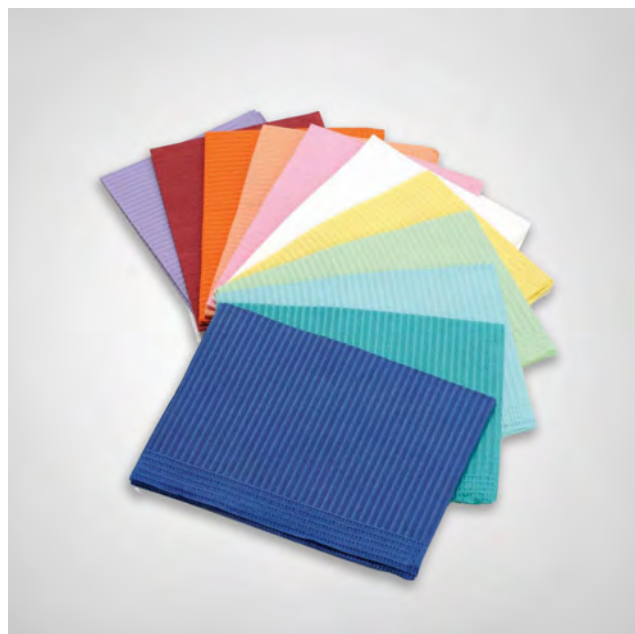
500 pz per scatola
500 pcs | carton

PERSONALIZZABILE CUSTOMIZABLE

In base alla richiesta del cliente
According to the customer's request

DIMENSIONI SIZE

33x45 cm

KIT COPRIRIUNITO IN TNT DENTAL CHAIR COVER KIT

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Sogeva universal dental unit covers kits are made of soft 30gr nonwoven fabric with high filtration and absorption properties. They are anatomical, with elastic for fastening and adhere perfectly to every type of dental unit.

Each DENTAL CHAIR COVER KIT contains three pcs:

- HEAD REST COVER
- BACK REST COVER
- CHAIR COVER

Possibility to purchase head rest cover and back rest+chair cover separately.

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Crepe paper in a small size to place on trays, specially designed to undergo the autoclave cycle. The product meets the requirements for steam sterilization (134°C), ethylene oxide and gamma rays.

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Disposable trays made of durable reinforced plastic material (polystyrene), non-sterile, biodegradable and with reinforced edges. They are used to rest instruments and materials.

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Containers in polypropylene suitable for the storage and transport of orthodontic plates, bites and functional appliances. They are reusable and can be sanitized with cold solutions.

- ORTHO BOX: height 40mm | Satined
- ORTHO BITE: height 30mm | Lucid

They are made of non-toxic food-grade plastic, free of latex and phthalates.

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COLORI COLORS

Verde neon, neon fucsia, arancio neon, giallo neon, arancio, bianco, verde, nero, fucsia, rosso, blu, azzurro, grigio, lime, lilla.

Neon green, neon fucsia, neon orange, neon yellow, orange, white, green, black, fucsia, red, blue, light-blue, grey, lime, lilac.

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Orthodontic container and denture holder for cleaning all types of dentures and appliances. Suitable for holding an upper and lower denture at the same time and equipped with a filter for impurities for cold.

They are made of non-toxic food-grade plastic, free of latex and phthalates.

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Green, light blue, lilac, neon fucsia, fucsia, red, orange, yellow, white, blue.

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Disposable cups in polypropylene, non-toxic material, with rounded edge and capacity of 180 cc.

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The disposable latex sheath, natural and biodegradable, allows repairing the surface of instruments used by the dentist in daily practice by considerably reducing bacterial and organic matter. It constitutes the ideal barrier against the risks of cross-contamination. It is extremely elastic and adaptable to objects of different shapes.

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Our latex and nitrile examination gloves are disposable and ambidextrous with smooth or rough surface with or without powder and with high tactile sensitivity and inherent softness that allows superior comfort and natural fit while avoiding hand fatigue even after prolonged use. They have a reinforced anti-roll edge at the wrist that ensures easy donning.

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Non-woven protective gown with ties at the waist and behind the back - Hydrophobic, antistatic, Autoclavable once before use.

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Saliva cotton rolls, non-sterile, disposable.
Strong absorbent power and resistance
to liquids.
Free of latex and phthalates

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Endodontic sponges are made by
polyurethane foam suitable for food contact
and autoclavable up to 134° C.
The product is disposable and comes in a
non-sterile condition. Intended for cleaning
root canal instruments after treatment.

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Adhesive film in roll for the protection of sleeves, panels, lights, air/water syringes.

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1200 strappi pre-tagliati in un dispenser
1200 pre-cut sheets packed in one dispenser box

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The multi-layer decontaminating mat consists of 30 numbered, removable, single-sided adhesive sheets containing a special formula of a bactericidal gelatine that kills the bacteria present on the soles of shoes. Its use is intended for all environments where the operator works with the patient and where therefore the total absence of contaminants and the maintenance of a high level of hygiene is required.

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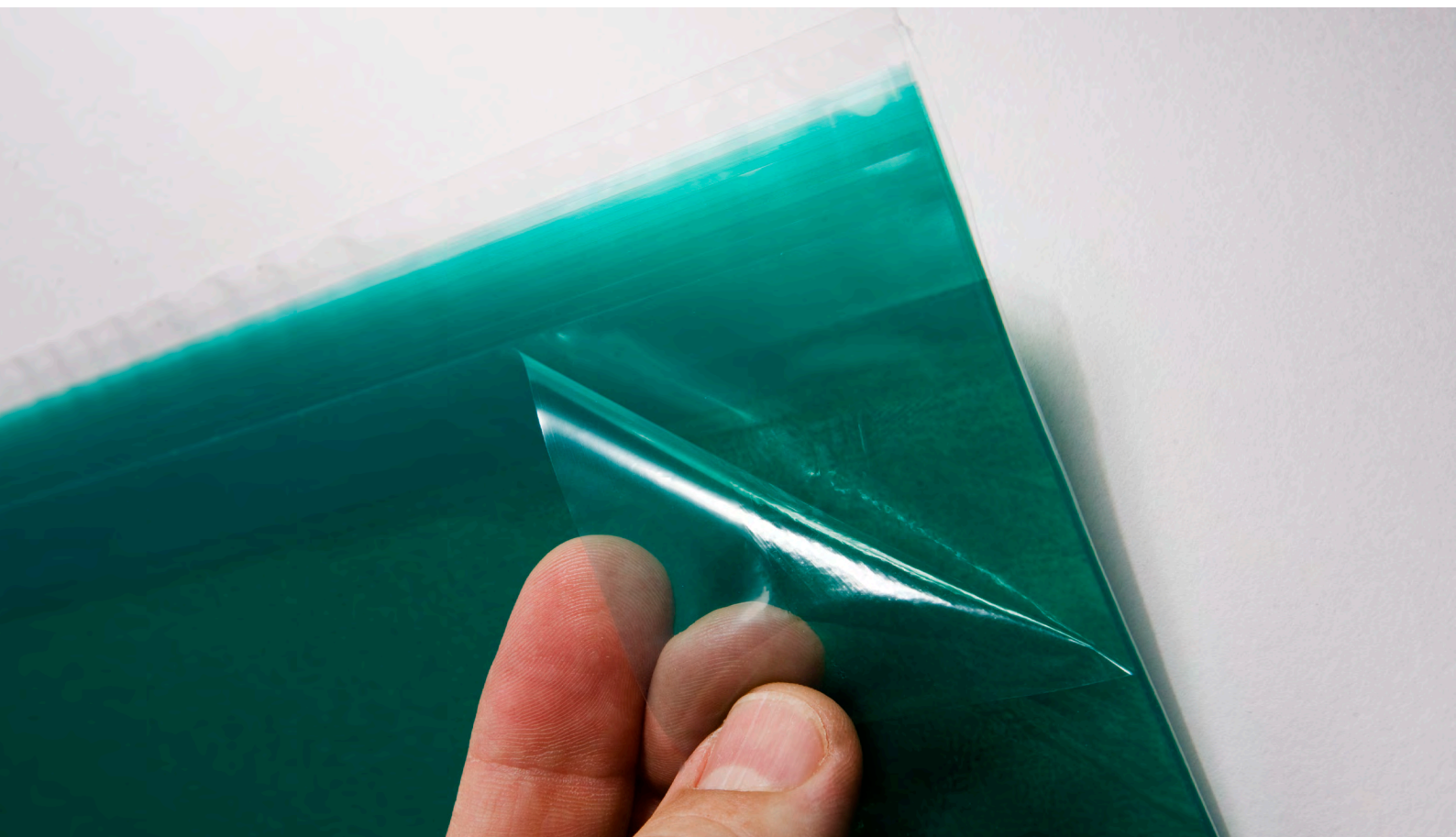
Verde, blu.
Green, blue.

PERSONALIZZABILE CUSTOMIZABLE

In base alla richiesta del cliente
According to the customer's request

THE **BEST PARTNER** FOR YOUR
STERILIZATION & MEDICAL ISSUES

+39 0302130958 | export@sogevasrl.com | commerciale@sogevasrl.com

www.sogevasrl.com